

You may also want to get these accessories for DisCide ULTRA gallon bottle:

Hold-It™ Gallon Bottle Holder

Make any gallon bottle easy to handle and store with our white metal holder. Mount as needed for easy access to bottle. Mounting hardware included.
Size: 5½" x 4½" x 4½"
Order No. 3518

Topcat™ Dynamic 1-Touch™ Gallon Sprayer and Holster

Can cover larger areas with one-handed ease. The holster attaches to the neck of your one-gallon bottle, instantly turning it into the perfect power tool for any job. The no-drip nozzle adjusts from spray to stream and anything in between. 4 AA batteries included.
Order No. 4230

DisCide Ultra is also available in:

- 60 count, 10½" x 10" Towelettes: Order No. 10DIS
- 160 count, 6" x 6¾" towelettes: Order No. 60DIS
- Quart bottle with sprayer: Order No. 3565Q
- 4 oz. bottle with pump sprayer: Order No. 3564

To order these or any other Palmero Health Care Products, call

1-800-344-6424
or visit
www.palmerohealth.com

REORDER #3565G

DisCide® ULTRA

DISINFECTING SPRAY

A One-Step Disinfectant - Cleaner - Decontaminant

Kills All Listed Pathogens IN ONE MINUTE

- ◆ Bactericidal
- ◆ Tuberculocidal
- ◆ Virucidal*
- ◆ Fungicidal
- ◆ Herpeccidal

DisCide ULTRA Disinfecting Spray is herbal scented, dual chain quaternary ammonium/high level alcohol solution. Use in Health Care & Alternate Care Settings such as hospitals, intensive care units, emergency medical settings, central supply, laboratories, clinics, nursing homes, and dental suites where control of the cross-contamination hazards between treated surfaces is required. Consumer Use on hard, non- food-contact surfaces in kitchens, food service areas and bathrooms and other related hard, non- food-contact surfaces such as shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, and door knobs and handles. DisCide ULTRA Disinfecting Spray eliminates odors and is effective against the following pathogens on hard non-porous, contact surfaces when used as directed:

- Tuberculosis bacteria (Mycobacterium bovis) (1 min. at 20°C)
- MRSA (Methicillin Resistant Staphylococcus aureus) (1 min.)
- Staphylococcus aureus (Staph) (1 min.)
- VRE (Vancomycin Resistant Enterococcus faecalis) (1 min.)
- Pseudomonas aeruginosa (1 min.)
- HBV* (Hepatitis B Virus) (1 min.)
- Salmonella enterica (1 min.)
- HCV* (Hepatitis C Virus) (1 min.)
- Escherichia coli (E. coli) (1 min.)
- RSV* (Respiratory Syncytial Virus) (1 min.)
- Avian Influenza A Virus* (H3N2 strain) (30 sec.)
- HIV-1* (AIDS virus) (1 min.)
- Influenza A Virus* (Hong Kong strain) (30 sec.)
- Human Coronavirus* (30 sec.)
- Herpes Simplex Virus type 2* (genital herpes virus) (1 min.)
- Adenovirus type 2* (1 min.)
- Candida albicans (1 min.)
- Pandemic 2009 H1N1 Influenza A Virus* (30 sec.) (formerly called swine flu)
- Proteus mirabilis (1 min.)

DisCide ULTRA Disinfecting Spray meets the disinfection requirements of OSHA's Bloodborne Pathogen Standard (29 CFR 1910.1030)

NET CONTENTS: 1 U.S. GALLON (3.79L)

ACTIVE INGREDIENTS

n-alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chloride 0.12%
n-alkyl (68% C12, 32% C14) dimethyl ethyl benzyl ammonium chloride 0.12%
Isopropyl alcohol 63.25%
OTHER INGREDIENTS 36.51%
TOTAL INGREDIENTS 100.00%

KEEP OUT OF REACH OF CHILDREN WARNING

See side panel for additional precautionary statements

EPA REG. NO. 10492-5 • EPA EST. NO. 62511-KS-1



FIRST AID

- If in eyes:**
- Hold eye open and rinse slowly and gently with water for 15-20 minutes.
 - Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
 - Call a poison control center or doctor for treatment advice.

HOT LINE NUMBER

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact Chem-Tel Inc. at 1-800-255-3924.

Precautionary Statements:

Hazards to Humans and Domestic Animals: WARNING. Causes substantial but temporary eye injury. Do not get in eyes or on clothing. Wear protective eyewear. Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash clothing before reuse. **Physical or Chemical Hazards:** Flammable. Keep away from heat or open flame

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal. **PESTICIDE STORAGE:** Store away from heat. **CONTAINER DISPOSAL:** Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available.

For technical information on DisCide ULTRA Spray, Towelettes, Aerosols and DisCide XRA Hand Sanitizing Wipes, Call 1-800-344-6424 or fax 203-377-8988.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling
GENERAL PRECAUTIONS AND RESTRICTIONS: This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contact intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection. **TO CLEAN, DECONTAMINATE, AND DISINFECT:** Remove gross dirt and soil. Thoroughly wet the surface with DisCide ULTRA Disinfecting Spray and allow to remain wet for the required contact time.
Healthcare FOR DISINFECTION AND DEODORIZATION OF: inanimate, hard, non-porous surfaces such as chairs, stools, counter tops, drawer pulls, door knobs and handles, carts, baskets, tables, cabinets, lighting fixtures, bassinets, bed rails, incubators, physical therapy equipment, shower stalls, toilet exteriors and toilet seats, telephones and other related hard surfaces in hospitals, operating and surgical rooms, intensive care and critical care units, emergency units, dental and veterinary offices, surgery, recovery, anesthesia, X-Ray, Cat. Lab., blood collection and donation centers, isolation units, orthopedics, newborn nursery, respiratory therapy, radiology, pediatrics, physical therapy, rehabilitation, ophthalmology, optometry, hospices, central supply, emergency medical vehicles, police and fire vehicles, correctional facilities, funeral homes, morgues, spas, health clubs, salons, schools and school medical facilities, school buses, day care and non-food contact surfaces. Use on stainless steel, glass, formica and other similar surfaces. Use care when using on varnished surfaces or plexi-glass. DisCide ULTRA Disinfecting Spray may be used to preclean or decontaminate medical instruments prior to sterilization or high-level disinfection.
Consumer FOR DISINFECTION AND DEODORIZATION OF: inanimate, hard, non-porous kitchen/food service and bathroom surfaces such as counter tops, drawer pulls, carts, baskets, tables, cabinets, door knobs and handles, appliance exteriors, shower stalls, toilet exteriors and toilet seats, bathroom and lighting fixtures, sinks, flooring, walls, vanities, diaper pails, trash cans, telephones, shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, and other related hard, non-food-contact surfaces. Use on stainless steel, glass, formica and other similar surfaces. Use care when using on varnished surfaces or plexi-glass.
***Virucidal Activity:** DisCide Ultra Disinfecting Spray is effective against the following viruses in 30 seconds or one minute when used according to label directions on hard, non-porous environmental surfaces: MRSA, VRE, Avian Influenza A (H3N2 strain), Influenza A (Hong Kong strain), Pandemic 2009 H1N1 influenza A virus (formerly called swine flu), RSV, HBV, HCV, Human coronavirus, HIV-1, Adenovirus type 2, and Herpes Simplex virus type 2. Kills HIV-1, HBV, and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV), and Hepatitis C Virus (HCV).
SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS), HBV, and HCV FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS.
PERSONAL PROTECTION: Wear disposable latex gloves, protective gown, face mask and eye covering, as appropriate, when handling HIV-1, HBV, and HCV infected blood or body fluids. **CLEANING PROCEDURE:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant. DisCide ULTRA Disinfecting Spray can be used for this purpose. **CONTACT TIME:** Thoroughly wet surface with DisCide ULTRA Disinfecting Spray. Allow surfaces to remain wet for 1 minute, then air dry. The efficacy of a 1 minute contact time has been shown to be adequate against HIV-1 (AIDS virus), HBV, and HCV. **DISPOSAL OF INFECTIOUS MATERIALS:** Blood and other body fluids should be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PALMERO HEALTH CARE
120 Goodwin Place, Stratford, CT 06615 • 800-344-6424 • www.palmerohealth.com PAL3565G-1006



BAR CODE #		
UPC is at the following percentage: 100%		
CHECK THE FOLLOWING AND INITIAL	APPROVED	NOT APPROVED
UPC shows the correct numbers.		
Bottle Type:		
Correct Copy		
Design Layout		
Cutoff measurement		
Layflat measurement		
Colors		
<input type="checkbox"/> Laser Approved <input type="checkbox"/> Laser Approved with change <input type="checkbox"/> Resubmit New Laser With Revisions		
NOTES:		

Signature: _____

Date: _____

Please Return To:

Plastic Packaging, Inc.
1246 Main Ave. SE
Hickory, NC 28602
(828) 328-2466
email:

3.125"

6.75"

3.125"

6.75"

LAYFLAT 9.875"

LAYFLAT 9.875"

Printing colors

- new match blue
- Pantone 465 C
- White

Non-printing colors

- Plastic Packaging logo - DO NOT PRINT
- clear - DO NOT PRINT
- die - DO NOT PRINT

CUTOFF 5.625"

CO