

- **\*Virucidal**
- **Tuberculocidal**
- **Bactericidal**
- **Pseudomonicidal**
- **Bleach Free**
- **Alcohol Free**



# Disinfectant Wipes

**Cleans, Disinfects & Deodorizes**

*This is NOT a baby wipe!  
Not for personal cleansing.  
Do not flush in toilet.*

**NET CONTENTS:  
2 LBS., 1.89 OZ.**

**160 Pre-moistened Wipes (6" x 6.75")**

**ACTIVE INGREDIENTS:**  
Alkyl (60% C14, 30% C16, 5% C12, 5% C18)  
dimethyl benzyl ammonium chlorides..... 0.105%  
Alkyl (68% C12, 32% C14) dimethyl  
ethylbenzyl ammonium chlorides..... 0.105%  
**INERT INGREDIENTS:**..... 99.790%  
**TOTAL:**..... 100.000%

**KEEP OUT OF REACH OF CHILDREN**  
**CAUTION:** SEE SIDE PANEL FOR ADDITIONAL  
PRECAUTIONARY STATEMENTS.

LYSOL® Brand I.C.™ Disinfectant Wipes are intended for use in :

- Hospitals and healthcare settings: ambulances, anesthesia, CAT labs, Central Supply/Clinics, Dental Offices, Dialysis Clinics, Doctor's offices, Emergency Rooms, Home health care, Hospices, Laboratory, Laundry rooms, Newborn nursery, Nursing homes, Pediatrics, Physical therapy, Physician's offices, Radiology, Recovery Room, Rehabilitation, Respiratory therapy, Transport vehicles, X-Ray.
- Critical Care Areas: CCU, Emergency Rooms, E.R., I.C.U., Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU), Surgery, Ophthalmic offices.
- Schools, Nursery, Daycare centers, health clubs, gyms, athletic facilities.
- Grocery stores, Shopping Centers, Delicatessens, Catering facilities, Banquet halls.
- Airports, transportation terminals.
- Veterinary clinics, animal hospitals, animal life science laboratories, kennels, pet animal kennels, breeding establishments, grooming establishments, pet animal quarters, zoos, pet shops, tack shops.
- Animal holding areas: veterinary waiting rooms, veterinary examination rooms, veterinary X-Ray rooms, veterinary operating rooms, animal transportation vehicles.



LYSOL® Brand I.C.™ Disinfectant Wipes will clean and disinfect hard, non-porous surfaces of : bed railings, cabinets, carts, chairs, counters, exam tables, gurneys, isolettes, IV poles, stethoscopes, stretchers, tables, telephones, toilet seats, toilets, bathtubs, bathroom fixtures, shower stalls, sinks, tiles, vanity tops, hospital beds, traction devices, MRI, CAT, scales, paddles, and wheelchairs.

LYSOL® Brand I.C.™ Disinfectant Wipes will clean and disinfect hard, non-porous surfaces of: gym equipment, exercise machines, wrestling mats, and gymnastic equipment.

LYSOL® Brand I.C.™ Disinfectant Wipes will clean and disinfect hard, non-porous surfaces of: grocery cart handles, grocery carts, shopping carts, grocery cart child seats.

**DIRECTIONS FOR USE:**  
It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

**CLEANING:** Use one LYSOL® Brand I.C.™ Disinfectant Wipes to completely remove surface of all gross debris from surfaces to be treated.

**DISINFECTING:** To disinfect hard, non-porous surfaces, thoroughly wet the surface to be treated. Treated surface must remain visibly wet for disinfection:

- To disinfect against *Escherichia coli* (E. coli), *Escherichia coli* O157:H7, *Pseudomonas aeruginosa*, *Salmonella choleraesuis*, Vancomycin Resistant *Enterococcus* (VRE), *Mycobacterium bovis* BCG (Tuberculosis), *Staphylococcus aureus*, Methicillin Resistant *Staphylococcus aureus* (MRSA), and Vancomycin intermediate Resistant *Staphylococcus aureus* (VISA), allow the treated surface to remain wet for 10 minutes.

A potable water rinse is required for surfaces in direct contact with food.

For plastic and painted surfaces, spot test on an inconspicuous area before use.

For heavily soiled surfaces, clean before following disinfecting instructions.

**\*VIRUCIDAL ACTIVITY:** LYSOL® Brand I.C.™ Disinfectant Wipes is an effective virucide on hard, non-porous surfaces against:

- Human Immunodeficiency Virus Type 1 (HIV-1) when the treated surface is allowed to remain wet for 2 minutes.
- Norovirus (Norwalk Virus) and Feline calicivirus when the treated surface is allowed to remain wet for 10 minutes.

Use additional wipe if needed to assure that the treated surface remains continuously wet for the prescribed time.

**KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.**

**PERSONAL PROTECTION:** When handling items soiled with blood or body fluids use disposable latex gloves, gowns, masks and eye coverings.

**CLEANING PROCEDURES:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product.

**CONTACT TIME:** Thoroughly wet surface with a new wipe. Allow surface to air dry. Although efficacy at a 2 minute contact time has been shown to be adequate against HIV-1, use the contact time for the disinfection of other organisms listed on this label.

**DISPOSAL OF INFECTIOUS MATERIALS:** Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

Not for personal cleansing. This is **not** a baby wipe! Do not flush in toilet.

**DISPENSER:** To start feed, remove large cover and discard seal from container. From center of roll, pull up wipe corner, twist into a point and thread it through the hole in the container cover. DO NOT PUSH FINGER THROUGH HOLE. Pull through about one inch. Replace cover. Pull out first wipe and snap off at a 90 degree angle. Remaining wipes feed automatically ready for next use. When through using, keep small center cap closed to prevent moisture loss. Do not flush.

0285326

**PRECAUTIONARY STATEMENTS**

Hazards to Humans and Domestic Animals  
**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

<b>FIRST AID</b>	
Have the product container or label with you when calling a Poison Control Center or doctor, or going for treatment.	
<b>If in eyes</b>	<ul style="list-style-type: none"> <li>• Hold eyes open and rinse slowly and gently with water for 15-20 minutes.</li> <li>• Remove contact lenses, if present, after the first 5 minutes, then continue rinsing.</li> <li>• Call a Poison Control Center or doctor for treatment advice.</li> </ul>
<b>CONTACT INFORMATION</b>	
You may contact 1-800-677-9218 for emergency medical treatment information.	

**STORAGE AND DISPOSAL**

Do not contaminate water, food or feed by storage or disposal.  
**PESTICIDE STORAGE:** Store in an area inaccessible to children.  
**CONTAINER STORAGE / DISPOSAL:** Wipes: Dispose of wipe in trash after use. Do not flush. Canister: Tightly close lid between uses to retain moisture. Nonrefillable container. Do not reuse or refill this container. Offer empty container for recycling if available or discard in trash.

**Questions? Call 1-800-677-9218**

For ingredient and other information,  
[www.rbnainfo.com](http://www.rbnainfo.com)  
Distributed by: Reckitt Benckiser Inc.  
Parsippany, NJ 07054-0225  
Made in U.S.A. © RBI 2010



EPA Reg. No.: 1839-174-675  
EPA Est. No.: 008251-WI-005;  
50757-WI-1; 8791-MO-1

