

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION

Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for further advice.

If swallowed: Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything to an unconscious person.

If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment.

If inhaled: Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth if possible. Call a poison control center or doctor for further treatment advice.

For emergency information on product, call the Customer Service Department at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time. After 4 p.m., call Chemtrec 24-Hour Emergency Service at 1-800-424-9300.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

STORAGE AND DISPOSAL

Towelette - Do not reuse towelette. Dispose of used towelette in trash.

Dispenser - Do not reuse empty container. Wrap container and discard in trash.

Distributed by:
PATTERSON
DENTAL SUPPLY, INC.

Corporate Office
1031 Mendota Heights Rd.
Saint Paul, MN 55120

MADE IN USA

80092-22E

70-7511-7



+D66207087720920

TUBERCULOCIDAL • BACTERICIDAL • VIRUCIDAL* • FUNGICIDAL

PATTERSON BRAND

pd CARE Wipes™

Disinfecting Towelettes

Net Contents: 160 Pre-Saturated Towelettes (6" x 6.75")

PD CARE Wipes Disinfecting Towelettes are non-woven disposable towelettes pre-saturated with PD CARE Surface Disinfectant. PD CARE Wipes are intended for use in health care settings such as dental offices, hospitals, laboratories, clinics, ophthalmic offices and veterinary facilities. PD CARE Wipes are effective against the following microorganisms on hard, non-porous surfaces when used as directed:

- **Mycobacterium tuberculosis var: bovis (BCG)**
- **Staphylococcus aureus**
- **Pseudomonas aeruginosa**
- **Salmonella choleraesuis**
- **Trichophyton mentagrophytes**
- **Methicillin Resistant Staphylococcus aureus (MRSA)**
- **Vancomycin Resistant Enterococcus faecalis (VRE)**
- **Hepatitis B Virus (HBV)***
- **Herpes simplex virus types 1 and 2***
- **Human Immunodeficiency Virus (HIV-1)***

ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride.....	0.233%
Isopropanol	14.300%
INERT INGREDIENTS:	85.467%
TOTAL	100.000%

EPA Reg. No. 46781-8-43100 • EPA Est. No. 56952-WI-001

KEEP OUT OF REACH OF CHILDREN

CAUTION

See back panel for additional precautionary statements.

Reorder Number

087-7209



DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

PD CARE Wipes Disinfecting Towelettes are non-woven disposable towelettes pre-saturated with PD CARE Surface Disinfectant. PD CARE Wipes are intended for use on hard non-porous surfaces in health care settings such as dental offices, hospitals, emergency medical settings, neonatal units, surgical centers, laboratories, clinics, nursing homes, ophthalmic offices, veterinary facilities, salon settings, emergency vehicles, correctional facilities, schools, health club facilities, non-food contact surfaces and other critical care areas where control of cross contamination is required. PD CARE Wipes will clean and disinfect, when used as directed, such items as tables, chairs, counter tops, carts, baskets, cabinets, lights, telephones, dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, exterior surfaces of infant incubators, exterior surfaces of anesthesia machines and respiratory therapy equipment which are made of stainless steel, glass, Plexiglas® and plastics (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene).

DO NOT USE AS A BABY WIPE.

Dispenser Directions:

To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

Cleaning and Disinfection Instructions:

Use one PD CARE Wipes towelette to completely preclean surface of all gross debris. To disinfect the precleaned surface, use a second PD CARE Wipes towelette to thoroughly wet the surface and allow to remain visibly wet for 3 minutes at 20°C. For tuberculocidal disinfection, allow the surface to remain visibly wet for 5 minutes at 20°C.

PD CARE Wipes effectively kills HIV and HBV on precleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) and Hepatitis B Virus (HBV).

Special Instructions for Cleaning and Decontamination Against HIV-1 (Human Immunodeficiency Virus) and HBV of Surfaces or Objects Soiled With Blood or Body Fluids:

Personal Protection: When using this disinfecting towelette, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A PD Care Wipes towelette can be used for this purpose.

Contact Time: Thoroughly wet surface with a new PD Care Wipes towelette. Allow to remain visibly wet for 3 minutes.

Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids should be autoclaved and disposed of in accordance with Federal, State and local regulations for infectious materials disposal.