

DIRECTIONS FOR USE - GENERAL CLASSIFICATION
IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

ProSpray is a ready to use surface disinfectant for use in the medical, dental, hospital, clinical, office and veterinary environments.

FOR DISINFECTION: Remove excess organic debris and thoroughly clean surface prior to application of ProSpray. The following non-porous surface areas: walls, floors and countertops should be sprayed and mopped, sponged or wiped in a manner to thoroughly wet the surface with ProSpray. Allow surface to remain wet for 10 minutes at 68°F (20°C) to destroy the following bacteria: Staphylococcus aureus, Escherichia Coli 0113, Salmonella typhosa, Pseudomonas aeruginosa, Mycobacterium tuberculosis var. bovis; the following fungi: Trichophyton mentagrophytes; the following viruses: *Herpes virus types 1 & 2, *Vaccinia virus, *Influenza virus types A1 (New Jersey), *Influenza B (Maryland) and *HIV-1 (AIDS virus).

KILLS HIV ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/ BODY FLUIDS in health care settings or other setting in which there is an expected likelihood of soiling inanimate surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

STORAGE AND DISPOSAL
Do not contaminate water, food or feed by storage or disposal.

PESTICIDE STORAGE: Store at room temperature.

PESTICIDE DISPOSAL: Wastes resulting from the use of this product may be disposed of on site or at an approved waste disposal facility.

CONTAINER DISPOSAL: Rinse and discard empty container. Do not reuse empty container.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids, always wear disposable latex gloves, gowns, masks, and eye coverings.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant.

DISPOSAL OF INFECTIOUS MATERIAL: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

CONTACT TIME: Leave surface wet for 2 minutes. Use a 10 minute contact time for disinfection against all other virus, bacteria, and fungi claimed.

Lemon
Scent

CERTOL®
INTERNATIONAL, LLC

ProSpray™

Surface Disinfectant

Tuberculocidal, Virucidal*, Bactericidal, Fungicidal

Recommended for use in hospitals, medical/dental clinics, nursing homes, healthcare institutions and veterinary clinics.

Active Ingredients

o-phenylphenol 0.28%

o-benzyl-p-chlorophenol 0.03%

Inert Ingredients 99.69%

Total 100.00%

DIN: 02231483

E.P.A. Reg. No. 46851-5

E.P.A. Est. No. 46851-CO-003

Contents: 24 Ounces (710 ml) • Reorder No. PSC240

Keep Out of Reach of Children

▽CAUTION:

Do Not Take Internally

Irritant

See Side Panels for Additional Precautionary Statements

PRECAUTIONARY STATEMENT

HAZARD TO HUMANS AND DOMESTIC ANIMALS

CAUTION: Harmful if swallowed or absorbed through skin or inhaled. Avoid contact with skin, eyes or clothing. Avoid breathing spray mist. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Remove and wash contaminated clothing before reuse.

USER SAFETY RECOMMENDATIONS

Users should wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Users should remove clothing/PPE immediately if pesticide gets inside. Then wash thoroughly and put on clean clothing.

GENERAL PRECAUTIONS AND RESTRICTIONS

Follow application as a low-pressure or high-pressure spray, do not enter or allow others to enter the treated area until sprays have dried. For all spray applications: Do not apply this product in a way that will contact workers or other person, either directly or through drift. Only protected handlers may be in the area during applications. For pour, pump, mop and wipe applications: Do not apply this product in a way that will contact other persons. Only protected handlers may be in the area during application.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water 15 - 20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a Poison Control Center or doctor for treatment advice.

If swallowed: Call a Poison Control Center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a Poison Control Center or doctor.

Do not give anything by mouth to an unconscious person.

If on skin or on clothing: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15 - 20 minutes. Call a Poison Control Center or doctor for treatment advice.

If inhaled: Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration preferably mouth-to-mouth if possible.

Call a Poison Control Center or doctor for further treatment advice.

HOTLINE NUMBER

In case of an emergency call toll free 1-800-535-5053. Have the product container or label with you when calling a Poison Control Center or doctor or going for treatment.



+H816PSC240F

Importé au Canada par/
Imported to Canada by:
Trans Canada Distribution Inc.
Mississauga, ON
L5L 5Y7

Manufactured by:
CERTOL®
INTERNATIONAL, LLC
6120 E. 58TH AVE.
COMMERCE CITY, CO 80022 USA
1 800 THE EDGE
(1 800 843 3343)
www.certol.com

PSC240/UL03 08/07